

Thank you for your order.

Test Submission Checklist

Following these instructions will avoid delays in the testing process.
Test start will be delayed if the saliva sample is improperly prepared.

Follow collection steps on reverse side of this form:


- Spit saliva into the empty blue container (Figure A on reverse).
- Spit until the amount of liquid saliva (not foam) reaches the level shown in (Figure B on reverse).
- Cap the blue container with the white lid (Figure C on reverse).
- Tighten firmly and mix gently (Figure D of reverse).
- Place the container in the enclosed plastic biohazard bag, peel off the red adhesive protection strip and seal..

Shipping:

- Place the following items in the pre-addressed FedEx return mailer:
 - Plastic biohazard bag with specimen container enclosed per above.
 - Sign, date and enclose disclaimer.
- You can track the status of your mailer by logging onto FedEx.com and entering your tracking number on the FedEx return mailer.
- Call Federal Express at 1-800-GO-FEDEX (1-800-463-3339) for package pickup or drop off at any FedEx collection site.

Note: Your identification is maintained throughout the genetic testing process by the ID number on the Oragene™ DNA Self-Collection Kit. Do not remove this label. It is affixed on the side of the blue container in advance of shipping to you.

Oragene™ DNA Self-Collection Kit User Instructions

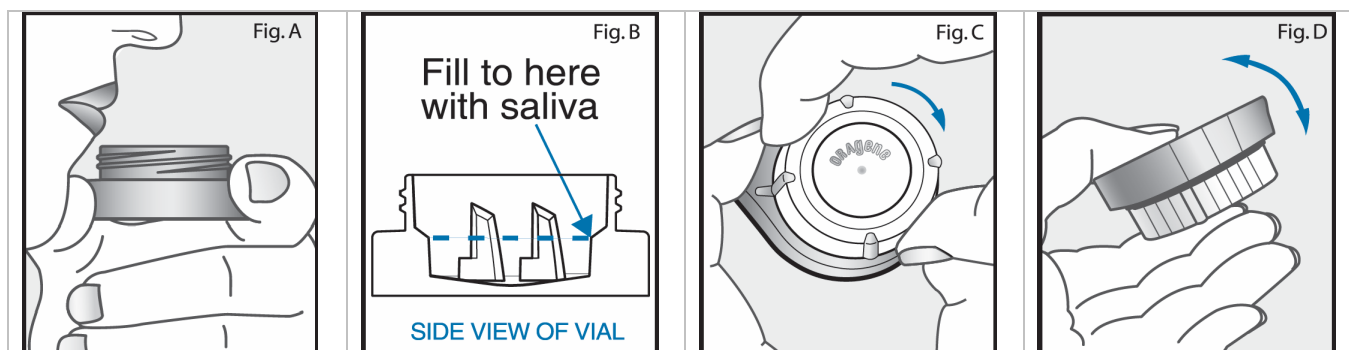

Before using, rinse your mouth with drinking water. After rinsing, discard or swallow drinking water. Then **wait 5 minutes** before spitting saliva sample in Step 1.

1 Spit saliva into the empty blue container (Figure A)	2 Spit until the amount of liquid saliva (not foam) reaches the level shown in Figure B	3 Cap the blue container with the white lid (Figure C)	4 Tighten firmly and mix gently (Figure D)
---	--	---	---

User Tips:

- Do NOT remove plastic film from the white lid.
- On average, it takes approximately 2 to 5 minutes to provide a saliva sample.
- Some people may find it hard to spit the recommended amount of saliva. To make more saliva, close your mouth and wiggle your tongue or rub your cheeks. It is easier to spit more if you place ¼ tsp of plain white sugar on your tongue.
- Finish spitting within 15 minutes and immediately close the container.

Intended Use: This product is designed for the safe collection of DNA samples from human saliva.

Contents: The white lid contains 2 mL of Oragene liquid. Before use, the solution in the lid should be clear and colorless.

Warnings: Do not ingest the Oragene liquid. Wash with water if the Oragene liquid comes in contact with eyes or skin.

Storage: Store at room temperature 15-30°C (59-86°F).